

Cleaning and Disinfecting the ComfortFull™ Mask




CLEANING

Clean the mask prior to using any of the disinfection or sterilization methods listed below.

- Remove the headgear and spacer, if appropriate.
- Soak the parts in a commercially available enzymatic cleaner (e.g., Klenzyme™).
- Clean the mask with a soft bristle brush in a solution of cool tap water and a commercially available anionic detergent (e.g., Manu Klenz™). Do not use cleaning products that contain conditioners or moisturizers because they will leave a residue.
- Rinse thoroughly under cool running tap water. Air dry.
- If adhering substances have not been adequately removed, replace the mask.

DISINFECTION/STERILIZATION PROCESS

WAVICIDE®

RESPIRONICS MASK	EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
ComfortFull 	45 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none"> • Inspect the mask after processing. If any components are damaged, replace the mask. • Slight discoloration of the mask cushion after processing is normal.

Contact Customer Service to obtain mask disinfecting instructions for other Respironics masks (#1011138).